

Lateral Flow Antigen Test Result Image Upload to Verify and Certify

1



Scan and open
the CoronaFocus
COVID TEST UK
App or webpage.



Download on the
App Store

2

Ensure you have sufficient internet
connection and a high-quality camera
on your mobile



Good Wi-fi



HQ Photo

3

Activate your Test Kit:
Log into your existing
CoronaFocus account.
New Customers use the
username and link sent
via email upon purchase.




4

Check the time frame of destination
travel rules. Perform swab sample as per
the instructions in the test kit.



5

Place cassette in the
marked area and wait 15
mins for the result to
appear. QR code, **person**
name and order
Booking Ref Code **must**
be clear and visible.

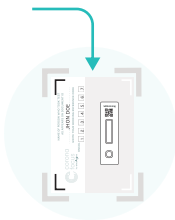

6

Photo must be taken after 15 mins and no
later than 30 mins after test was taken.



15-30
minutes

Confirm 

Confirm and submit result image via app

7

Set the swab date and time taken



8

Verified result and certificate notification
will be available via the app < 2 hrs

9

If you get a positive result, self isolate
and take a confirmatory PCR test



10



Incorrect

No name or
booking code

Correct

QR code, person name
and booking code visible



corona
focus
by oncológica

NAME OF PERSON WHO TOOK TEST
AS APPEARS ON DOCUMENT ID

WRITE YOUR BOOKING REFERENCE CODE

ONCOL

Place the cassette here

ENSURE PHOTO INCLUDES AREA WITHIN THE FRAME DETAILING PERSON NAME, BOOKING REFERENCE CODE AND QR CODE ON CASSETTE